
G O L D E N T C L A S S I C

M O U L D C H A R T

GOLDENT TEETH COMPLY WITH ISO 3336:1996 TYPE1 - TYPE2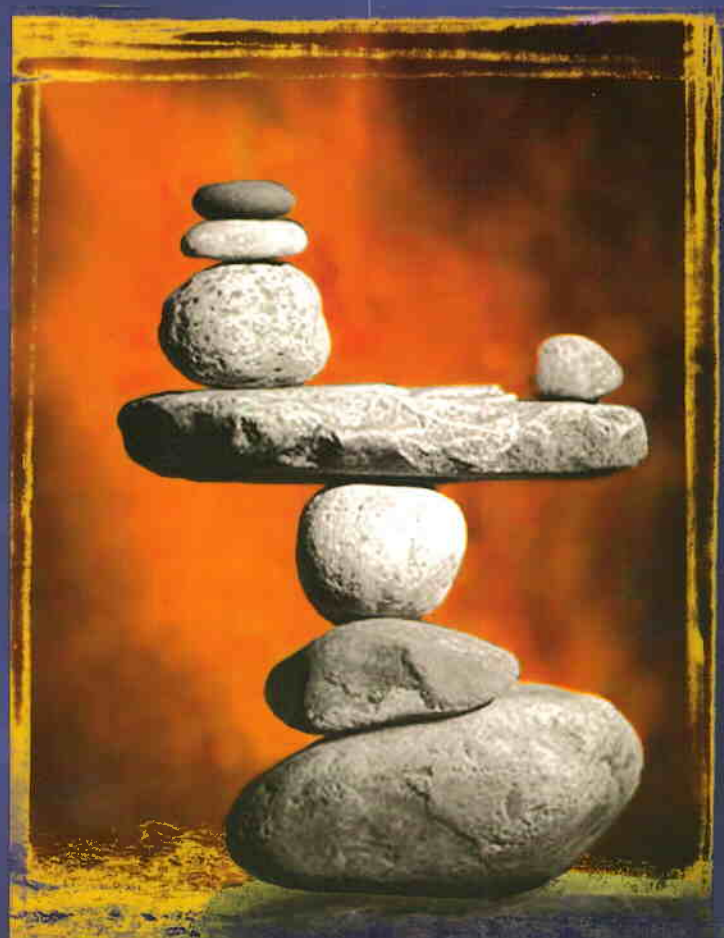




Goldent

ACRYLIC TEETH

Creating durability

Goldent

acrylic teeth mould chart

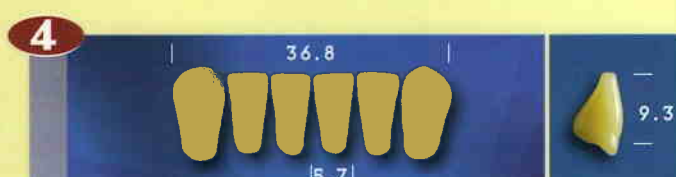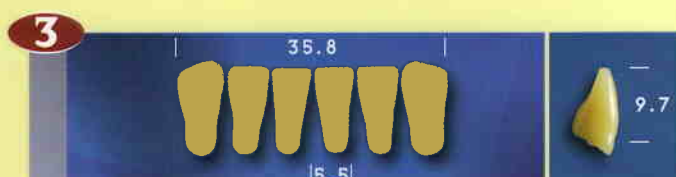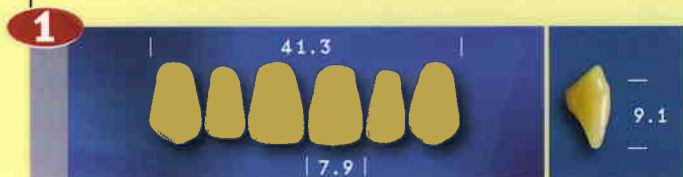The cross-linked Goldent acrylic teeth could be ordered in 16 different colour shades according to the standards of vita, from type A1-D4.

The cross-linked GOLDENT CLASSIC acrylic teeth are produced in 17 shapes (7 upper incisors, 4 lower incisors, 3 lower-upper molars) in 16 different colour shades according to the standards of vita.

The well established manufacturing technology and excellent raw material quality are guarantees for adequate mechanical and non-fading colour product features. With the great hardness of the occlusal surface and the inalterability of the colours, we are able to achieve excellent results in removable prostheses as well as in partial and combined prostheses.

The transparent edge of incisors, the development of well-visible internal mamelons and the slight spots of enamel are all greatly increasing and contributing to the "natural look" of GOLDENT CLASSIC acrylic teeth.

CE marked and manufactured under ISO 9001 and ISO 13488 Quality Standard, and in accordance with EN ISO 3336 for artificial teeth.

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For further details and ordering information please contact your dealer:

Goldent

GB SYNTHETIC POLYMER TEETH FOR DENTAL PROSTHESIS

- Store in a clean, dry place away from light.
- To ensure optimum bonding to the denture base resin, grind teeth on the cervix prior to setting up, remove wax and degrease with a methyl methacrylate monomer.
- Direct the patient to adopt daily hygiene protocols, suggest him to avoid excessive brushing which could cause abrasive wear, and not to immerse in alkalis, acids or other substances that can damage acrylic materials.
- This product does not contain substances classified as hazardous conforming to Directive 67/548/EEC and subsequent amendments.
- Grinding of the teeth generates dust which is classified as „not otherwise classifiable“. Do not inhale. Use appropriate protection: TLV/TWA applies.
- Dispose conforming to local law.

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GOLDENT HUNGARY LTD.